




SCHNELL
MEDICAL




KAIMEN™

Ideal high absorbent sponge specially for laparoscopic surgery

- **Supports to secure dry surgical field by its superior absorbency**
Promptly absorbs blood, lymph fluid etc by simply leaving the sponge on the site.
Reduce stress of surgeons by providing clear surgical field.
- **4 sizes assorted pack**
Assort of 2/3/4/5cm pack for appropriate case and site of the patient.
- **X-ray detectable**

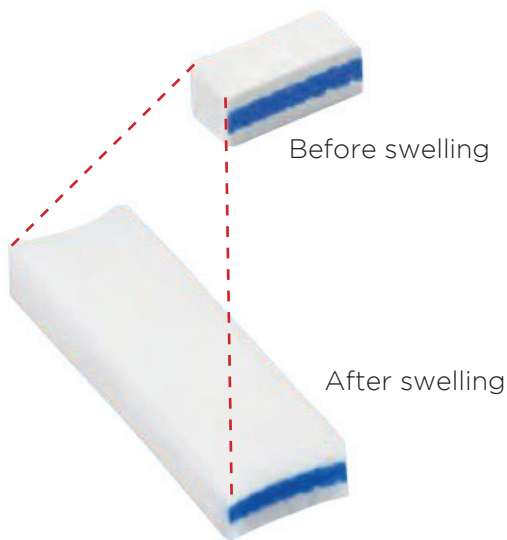


Example of use by Laparoscopic Assisted Distal Gastrectomy

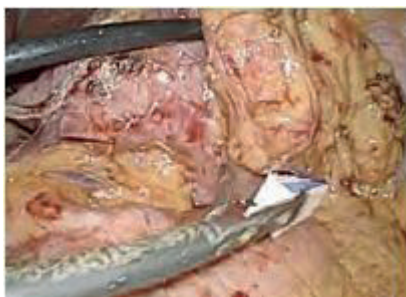

Gastric Corpus * dorsal




Caudate lobe of liver * dorsal

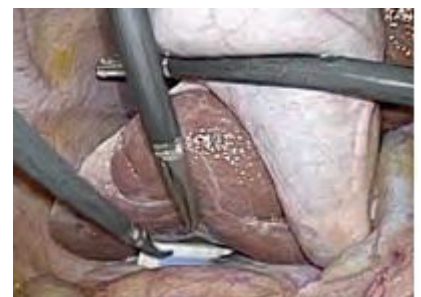

Morison's pouch
(hepatorenal recess)

KAIMEN™

1. Take out KAIMEN™ from the case cleanly.
2. Insert KAIMEN™ into body cavity through trocar (over 12mm dia.)
3. Simply detain the sponge in the part where lymph fluid and blood are easy to collect. The sponge swells by absorbing outskirts fluid.
4. Secure incision part over 2-3cm to take out swelled sponge from the bodycavity.
 - Remind that sponge is swelled and its size is enlarged.
 - Take out KAIMEN™ carefully not to be cut or broken.

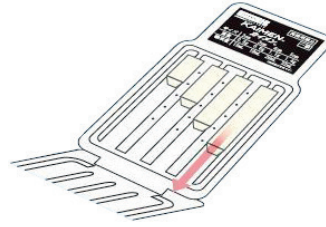

Size	Before swelling (HxWxD)	After swelling (HxWxD)	Absorbency Approx.
2 cm	8 x 8 x 20 mm	8 x 75 x 20 mm	13 mL
3 cm	8 x 8 x 30 mm	8 x 75 x 30 mm	19 mL
4 cm	8 x 8 x 40 mm	8 x 75 x 40 mm	26 mL
5 cm	8 x 8 x 50 mm	8 x 75 x 50 mm	32 mL

Order Information

Order Information

Code	Sizes	Quantity	Description
KM/088-650310	2/3/4/5 cm	4pcs/unit	KAIMEN